



## HAEMOSTASIS WORKSHEET 2010

The samples have been tested and found to be homogeneous and stable for the purpose of these exercises.

All samples are lyophilised plasma and will require reconstitution with distilled water. Please see over for reconstitution information.

<b>JANUARY</b>	Cycle 10 Run 1	INR10-01a INR10-01b APT10-01a APT10-01b	INR, Fibrinogen INR, Fibrinogen APTT, FVIII APTT, FVIII
<b>FEBRUARY</b>	Cycle 10 Run 2	INR10-02a INR10-02b APT10-02a APT10-02b	INR, Fibrinogen INR, Fibrinogen APTT, FVIII APTT, FVIII
<b>APRIL</b>	Cycle 10 Run 3	INR10-04a INR10-04b APT10-04a APT10-04b	INR, Fibrinogen INR, Fibrinogen APTT, FVIII APTT, FVIII
<b>MAY</b>	Cycle 10 Run 4	INR10-05a INR10-05b APT10-05a APT10-05b	INR, Fibrinogen INR, Fibrinogen APTT, FVIII APTT, FVIII
<b>JULY</b>	Cycle 10 Run 5	INR10-07a INR10-07b APT10-07a APT10-07b	INR, Fibrinogen INR, Fibrinogen APTT, FVIII APTT, FVIII
<b>AUGUST</b>	Cycle 10 Run 6	INR10-08a INR10-08b APT10-08a APT10-08b	INR, Fibrinogen INR, Fibrinogen APTT, FVIII APTT, FVIII
<b>OCTOBER</b>	Cycle 10 Run 7	INR10-10a INR10-10b APT10-10a APT10-10b	INR, Fibrinogen INR, Fibrinogen APTT, FVIII APTT, FVIII
<b>NOVEMBER</b>	Cycle 10 Run 8	INR10-11a INR10-11b APT10-11a APT10-11b	INR, Fibrinogen INR, Fibrinogen APTT, FVIII APTT, FVIII

## RECONSTITUTION INFORMATION

<b>SAMPLE</b>	<b>Reconstitution volume Distilled water</b>	<b>Test</b>	<b>Interpretation</b>	<b>Units</b>
INR10-**a / b	1 ml	INR	Below / Therapeutic / Above	Ratio
		Fib		g/L
APT10-**a / b	1 ml	APTT	Normal / Extended	sec
		FVIII	Normal / Abnormal	%

### **INR:**

- Reconstitute with 1ml of distilled water. Allow to stand at RT for 10 minutes before testing.
- Record results on Haemostasis result sheet 2010.
- INR - Interpretation should be in accordance with your THERAPEUTIC INTERVAL for anticoagulant therapy.
- Clinical information for each sample will be included with the dispatch insert.

### **Fibrinogen:**

- Test the reconstituted INR sample.
- Centrifuge samples prior to testing if a turbidimetric method is used.
- Record results and report in g/L on Haemostasis result sheet 2010.

### **APTT, FVIII:**

- Reconstitute with 1ml of distilled water. Allow to stand at RT for 10 minutes before testing.
- Record results on Haemostasis result sheet 2010.
- Consider these samples to be from patients being tested for pre-operative orthopaedic surgery.
- These patients are not on any anticoagulants.
- Mark the appropriate interpretation on the Haemostasis result sheet 2010.
- APTT – Interpretation should be in accordance with your reported REFERENCE INTERVAL.